

# Intoximeters

World Leader in Breath Alcohol Testing for Over Fifty Years

## F8 - GENERAL SETUP

**NOTE:** This function allows you to change the time on your EC/IR to and from Daylight Savings Time (DST). If you are using an EC/IR most units are programmed to automatically adjust to the 'Old' DST Format. If your instrument is programmed this way you will need to adjust the time on your unit four times per year based on the schedule noted below (i.e. New Format / Old Format).

**NEW FORMAT**  
DST Begins 2 a.m.  
(Second Sunday in March 2007)

**OLD FORMAT**  
DST Begins 2 a.m.  
(First Sunday in April)

DST Ends 2 a.m.  
(First Sunday in November)

DST Ends 2 a.m.  
(Last Sunday in October)

### TO CHANGE THE TIME ON YOUR EC/IR FOLLOW THE INSTRUCTIONS BELOW:

- 1) Instrument must be in *Normal Scrolling Mode*
- 2) Press the F8 key
- 3) Enter your password / Press Enter key
- 4) Continue to press the Enter key until unit displays: ENTER TIME: (HH:MM)  
XX:XX
- 5) Enter the correct time xx:xx (Military Time / 24-Hour Clock)
- 6) Press the Enter key until the unit returns to *Normal Scrolling Mode*
- 7) Verify the time is correctly displayed

<u>DISPLAY</u>	<u>KEYBOARD ENTRY</u>	<u>EXPLANATION</u>
1) Normal Scrolling Mode	2) Press the F8 key.	Starts the general setup program.
3) PASSWORD ****	Enter your password then press the Enter key.	This function requires a Technician level or higher. 4) Press the Enter key to scroll through the Menu.
DAY OF WEEK (1 - 7) TUESDAY	Enter the number corresponding to the day of the week by pressing the space bar or entering the corresponding number.	Day of week.  1=Sunday 2=Monday 3=Tuesday 4=Wednesday 5=Thursday 6=Friday 7=Saturday
ENTER DATE: MM/DD/YY 05/24/95	Enter the month, day, and last two digits of the year. 05 24 95 Do Not enter the "/". The instrument will format the date.	Enter the date in the month, day, year format. Preceding zeros need to be entered (e.g. 07/04/95)
5) ENTER TIME: (HH:MM) XX:XX	Enter the hour then minute: XX:XX.	Enter the time in the 24-hour/military time format i.e. 13:45 would be 1:45 PM. Preceding zeros need to be entered i.e. 01, 02, 08 etc. 6) Press Enter key until unit returns to <b>Normal Scrolling Mode</b>
Normal Scrolling Mode	Information only. No data entry required.	7) Verify correct date/time.

**If you have any questions contact Intoximeters Customer Service at 800-451-8639.**